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The screenshot shows the LabConnect patient management interface. On the left, there's a sidebar with '환자관리' (Patient Management) selected. The main area is titled '환자관리' and shows a table of patients with columns for Number, Registration Number, Patient Name, Gender, Birthdate, and Phone Number. A dropdown menu is open at the top right, containing options like '내정보수정' (Change Information), '비밀번호변경' (Change Password), '설정' (Settings), and '로그아웃' (Logout). The '비밀번호변경' option is highlighted with a red circle and a red box.

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The screenshot shows the '검사접수' (Exam Registration) page of the iKooB EMR system. At the top, there are navigation links for '환자관리', '검사접수', and '검사결과'. The main search area has a red box around it, labeled '1'. It includes fields for '검색조건' (Search Condition) with radio buttons for '내환자' (Inpatient) and '전체환자' (All Patients), and a text input field for '환자등록번호, 환자명, 휴대전화번호' (Patient Registration Number, Patient Name, Mobile Phone Number). There are also date pickers for '처방일' (Prescription Date) and '검사일' (Exam Date), separated by a minus sign. A blue '검색하기' (Search) button is to the right. Below the search area is a red box around the '검사결과' (Exam Results) table, labeled '2'. The table has columns for '번호' (Number), '처방일' (Prescription Date), '환자등록번호' (Patient Registration Number), '환자명' (Patient Name), '생년월일' (Birth Date), '휴대전화번호' (Mobile Phone Number), '검사항목' (Exam Item), '담당의' (Physician), '처방수량' (Prescription Quantity), '실시간측정' (Real-time Measurement), and '취소일' (Cancellation Date). The table contains four rows of data.

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
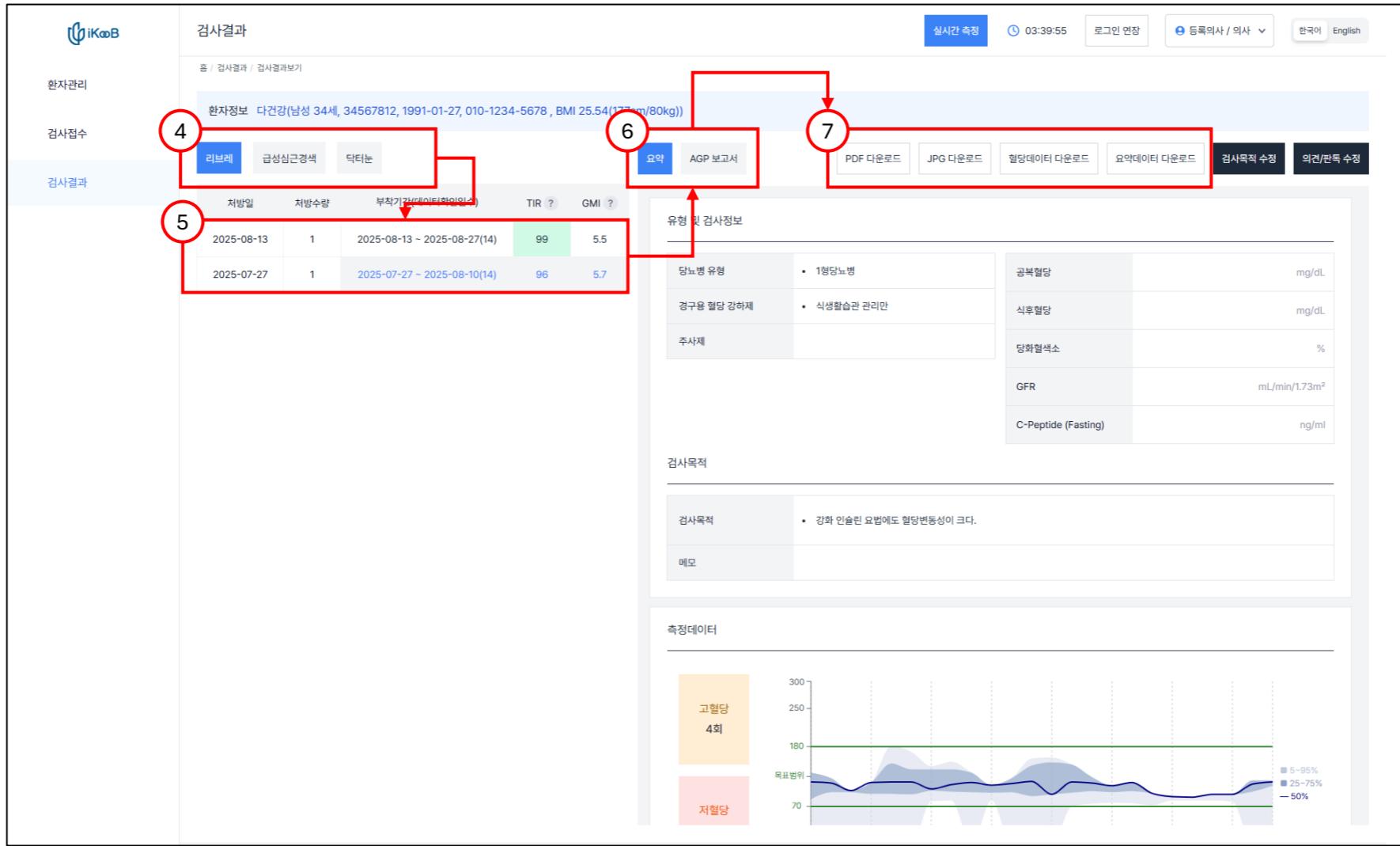
The screenshot shows the LabConnect platform's 'Exam Results' section for an outpatient visit. The interface includes a navigation bar with 'iKoB' and 'LabConnect' branding, a patient summary, and various menu options like '환자관리', '검사접수', and '검사결과'. A red box highlights the '검사결과' tab. Numbered callouts point to specific features:

- 4**: A red box highlights the '검사항목' (Exam Items) section, which includes buttons for '리브레' (Libre), '급성심근경색' (Acute Myocardial Infarction), and '당티논' (Dantizone).
- 5**: A red box highlights the '처방' (Prescription) section, showing two entries with dates, doses, and ranges.
- 6**: A red box highlights the '요약' (Summary) and 'AGP 보고서' (AGP Report) buttons.
- 7**: A red box highlights the 'PDF 다운로드' (PDF Download), 'JPG 다운로드' (JPG Download), '혈당데이터 다운로드' (Blood Glucose Data Download), '요약데이터 다운로드' (Summary Data Download), '검사목적 설정' (Set Exam Purpose), and '의견/판독 수정' (Modify Opinion/Reading) buttons.

The main content area displays '유형 및 검사정보' (Type and Test Information) with sections for '당뇨병 유형' (Diabetes Type), '경구용 혈당 강하제' (Oral Hypoglycemic Agents), and '주사제' (Insulin). It also includes a '검사목적' (Exam Purpose) section with a note about insulin therapy and a '측정데이터' (Measurement Data) section featuring a blood glucose graph with shaded confidence intervals and markers for '고혈당' (Hyperglycemia) and '저혈당' (Hypoglycemia).

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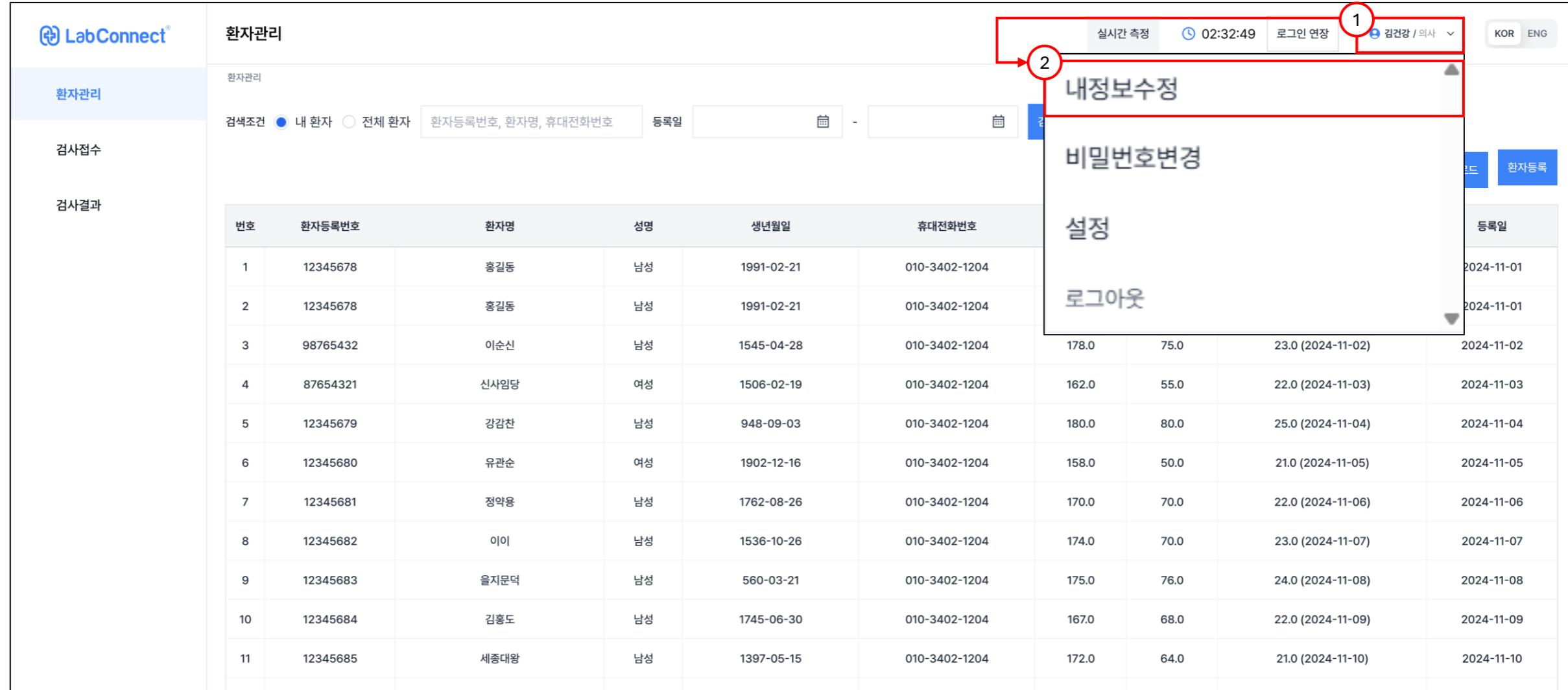The screenshot shows the LabConnect application interface. On the left, there's a sidebar with '환자관리' (Patient Management) selected. The main area displays a table of patient records. A red box highlights the top right corner of the screen, which contains the user profile information: 실시간 측정 (Real-time measurement), 02:32:49 (Timestamp), 로그인 연장 (Login Extension), 김건강 / 의사 (Kim Geonkyang / Doctor), KOR ENG language switch, and a dropdown menu. A red arrow labeled '1' points to the login extension button. Another red box highlights the '내정보수정' (Modify Information) option in a dropdown menu that appears when the user profile is clicked. A red arrow labeled '2' points to this option.

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The screenshot shows the LabConnect patient management interface. On the left, there's a sidebar with '환자관리' (Patient Management), '검사접수' (Exam Reception), and '검사결과' (Exam Results). The main area is titled '환자관리' and shows a list of patients with columns for '번호' (Number), '환자등록번호' (Patient Registration Number), '환자명' (Patient Name), and '등록일' (Registration Date). A red box highlights the '내정보수정' (Information Modification) button in the top right corner of the page.

A modal window titled '내 정보 보기' (View My Information) is open, containing fields for '병원명' (Hospital Name), '아이디' (ID), '병원아이디' (Hospital ID), '이름' (Name), '이메일' (Email), '휴대전화번호' (Mobile Phone Number), '진료과' (Specialty), and '면허번호' (License Number). A red box surrounds these fields, and a red arrow labeled '3' points to the '아이디' field. Another red arrow labeled '4' points to the blue '[수정]' (Modify) button at the bottom right of the modal.

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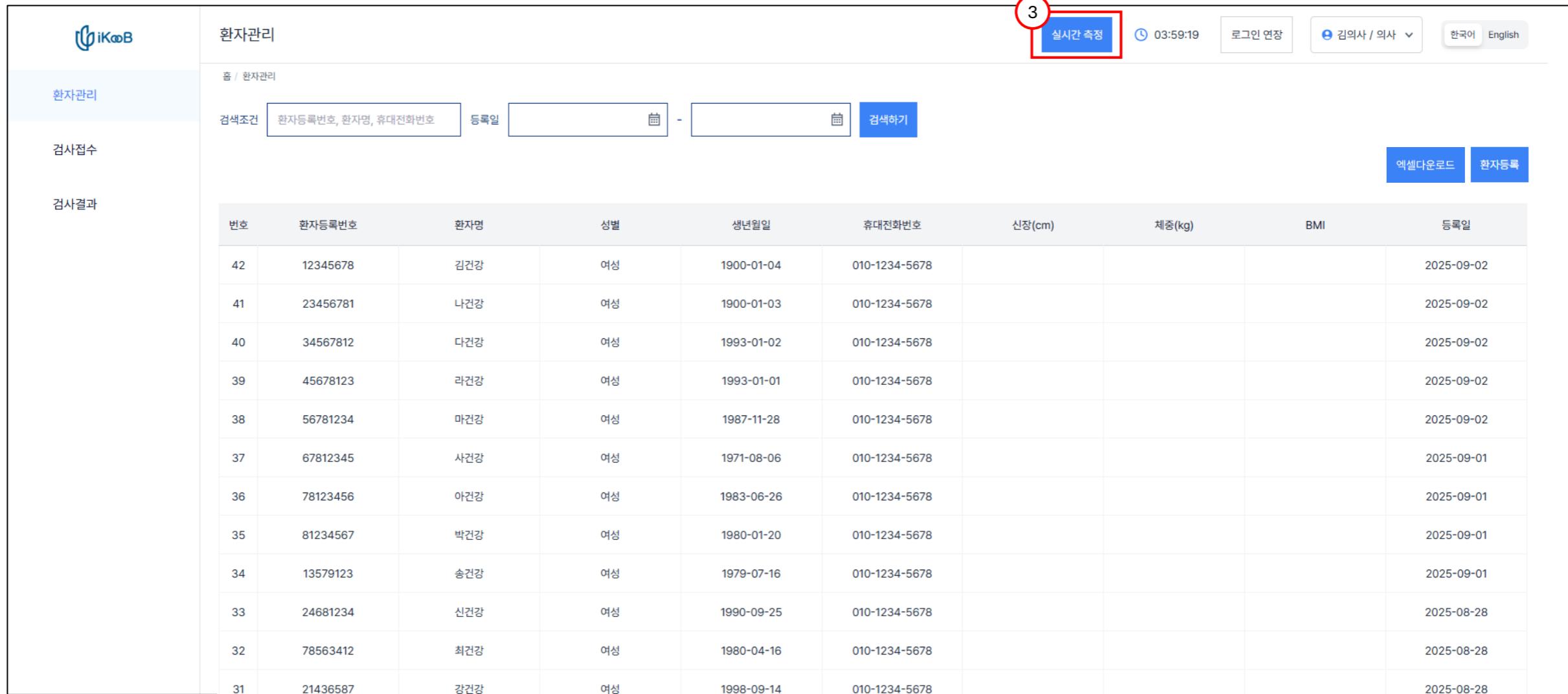
The screenshot shows the 'Patient Management' section of the iKoob LabConnect application. On the left, there's a sidebar with '환자관리' (Patient Management), '검사접수' (Sample Reception), and '검사결과' (Test Results). The main area is titled '환자관리' and shows a search bar with '검색조건' (Search Conditions) set to '환자등록번호, 환자명, 휴대전화번호' (Patient Registration Number, Patient Name, Mobile Phone Number). Below the search bar are two date pickers labeled '등록일' (Registration Date) and a '검색하기' (Search) button. To the right of the search bar are two buttons: '엑셀다운로드' (Excel Download) and '환자등록' (Patient Registration). At the top right, there are user status indicators: '로그인 연장' (Login Extension), '김의사 / 의사' (Dr. Kim / Doctor), and language options '한국어 English'. A red circle with the number '3' is drawn around the '실시간 측정' (Real-time Measurement) button, which is highlighted in blue.

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The screenshot displays the 'Real-time Monitoring (List Type)' feature of the CGM LiVE system. At the top, there are search and filter options: 'Entire Record' (1), 'Patient Search' (2), and 'Communication Status' (4). The main area shows a table of patient data with columns for Name, Record Number, Age, Gender, Hospitalization Date, Admission Date, Status, Blood Glucose Level, Confirmation Time, Product Name, and Discharge Date. Row 3 (highlighted by a red box) shows a patient named Park Sojini (박수진) with a checked checkbox (3).

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